# MEDISANA®

## Rollenmassage-Kissen RBM

Roll massage cushion RBM

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Almofada de massagem por rolos RBM

Rolmassage-kussen RBM

Rullahierontatyyny RBM

Rullmassage-kudde RBM

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- GB**
- 1 Neck support pillow
  - 2 Power supply unit
  - 3 Massage seat
  - 4 Arrangement of the massage zones  
4 rollers in the back area and  
2 vibrating massage motors in the upper  
thigh area
  - 5 Pocket for control unit
  - 6 Connector for power supply unit
  - 7 HEAT on/off button
  - 8 ROLL MASSAGE on/off button
  - 9 VIBRATING MASSAGE mode button  
- normal/pulse
  - 10 Control unit
  - 11 VIBRATING MASSAGE intensity button  
- low/high
  - 12 VIBRATING MASSAGE button - on/off
  - 13 ON/OFF button
  - 14 Control LED for "Unit on"
  - 15 Control LEDs for vibrating massage  
normal/pulse mode
  - 16 Control LED for "Heat on"
  - 17 Control LED for "Roll massage on"
  - 18 Control LEDs for vibrating massage  
low/high intensity

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**Thank you  
very much**

Thank you for your confidence in us and congratulations!  
You have acquired a **MEDISANA** quality product with this roll massage cushion **RBM**.

This unit is designed for massaging the back and upper thigh areas.

We recommend that you carefully read the following information on use and care of the product so that you experience its benefits and enjoy using your **MEDISANA** roll massage cushion **RBM** for many years.

**IMPORTANT**

**Please read the following safety information carefully before using the device and keep these instructions in a safe place for later use.**

**1.1  
About the power  
supply**

- Before connecting the device to your power supply, make sure that it is switched off and that the electrical voltage specified on the device's type label matches that of your mains power.
- Only connect the mains adapter whilst the unit is switched off.
- Do not operate this unit near high frequency electromagnetic sources.
- Keep the power cord and the device itself away from heat, hot surfaces, damp and liquids. Avoid touching the plug or power switch with wet or even damp hands, or when you are standing in water.
- Parts that conduct electricity on the device must not come into contact with liquid.
- If the device falls into water, do not try to pick it up. Instead pull the plug out immediately.
- Always make sure that the plug is easily accessible when the device is connected.
- Disconnect the plug from the socket immediately after using the device.
- Never pull on the power cord to disconnect the appliance from the mains. Always pull out the mains plug!
- Do not carry, pull or rotate the device by its power cord.

**1.2  
Specific groups  
of people**

- Special care is needed if the unit is used close to, by or in the vicinity of children, sick people or people who cannot help themselves.
- Never leave the unit unattended if it is connected to the power supply.
- Do not allow children to play with the unit. The unit must be kept away from children.
- Never allow children to play with the packaging film – risk of suffocation!

- Do not use this unit to supplement or replace medical treatment. Chronic complaints and symptoms may be made worse.
- The unit must not be used in the following cases: Tuberculosis, benign or malignant tumours, bleeding, skin irritation, phlebitis, thrombosis, open and fresh wounds, bruising, cracked skin, varicose veins and indeterminate calf pain.
- Do not use the unit if you have sensitive skin or heart/circulation problems, headaches or high blood pressure.
- Certain stress limits apply to pregnant women.
- Consult your doctor if you have any questions concerning the therapeutic use of massage cushions.
- If you have unexplained pain, are receiving treatment from a doctor and/or use medical equipment, consult your doctor before using the massage cushion.

### 1.3 Before using the unit

- Always carefully inspect the power supply, the cable, the control unit and the massage cushion for damage before use. Do not attempt to use a defective unit.
- Do not use the unit if there are visible signs of damage on the unit or the cable components, the unit does not work properly or the seat, the control unit or the mains adapter have been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.

### 1.4 Operating the unit

- The seat massager must only be used for its intended purpose as described in the operating instructions.
- Using the mat for any other purpose invalidates the warranty.
- Under no circumstances must accessories be used that are not explicitly recommended by the manufacturer.
- Do not allow the massage cushion to come into contact with pointed or sharp objects.
- Never put the unit down next to an electric stove or other sources of heat.
- Do not use the unit in places where aerosol (spray can) products or oxygen are being used.
- Do not stand on the unit.
- If you feel unwell or experience pain when using the unit, stop the treatment immediately.
- For understandable reasons, the manufacturer cannot be liable for damage caused by failure to comply with the operating instructions.
- If you give the the massage cushion to third parties, please ensure that they are also given the operating instructions.

**IMPORTANT**

The device is only intended for household use and must not be used for commercial or medical purposes. If you have health concerns, consult your doctor before using the massage cushion.

The seat massager must only be used in enclosed spaces!

Do not use the seat massager in damp rooms (e.g. when bathing or showering).

**1.5  
Maintenance  
and cleaning**

- Only use the mains adapter that is provided with the massage cushion.
- The unit is maintenance free. If a fault does occur, just check whether the mains power adapter is properly connected.
- If the unit becomes faulty, do not try to repair it yourself, since this invalidates the warranty. Have all repairs carried out by the service centre.

**IMPORTANT**

Do not wash!

Do not dry clean!

**2.1  
Scope of delivery  
and packaging**

Please check first whether the device is complete.

The package contains:

- 1 **MEDISANA** roll massage cushion **RBM**
- 1 Control unit
- 1 Power supply 230 V~ 50 Hz / 12 V~ 2500 mA
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

**WARNING**

Please ensure that the packing film is kept away from children!  
Risk of suffocation!

## 2.2 What is massage?

Massage is one of the oldest methods of treatment and an important part of physiotherapy. The mechanical treatment of the outer tissues of the body relaxes tense muscles, improves blood circulation and supports the regeneration of tissues.

Massage may also positively effect the vascular system, the inner organs and metabolism. There are six classic methods of massage: stroke massage, kneading, tapping, vibrating and roll massage as well as rubbing and slapping.

Furthermore, there are other important massage techniques such as manual lymphatic drainage, reflexology, underwater pressure massage, brush massage and finger pressure massage techniques (shiatsu, acupuncture).

## 2.3 How does the MEDISANA roll massage cushion work?

The **MEDISANA** roll massage cushion **RBM** offers two different massage techniques, vibrating massage in the seat and kneading massage by means of roller technology.

Two motors **4** in the seat cause it to oscillate at a high frequency, resulting in a vibrating motion that is transferred to the upper thigh. You have the option of choosing between two massage intensities (**Lo** low / **Hi** high) and between a normal, i.e. constant, or pulsating vibrating massage (**Normal** constant / **Pulse** pulsating).

Alternatively or at the same time, you can switch on the kneading massage function that operates by means of roller technology, with or without additional heat treatment from the back area of the massage cushion. Here four anatomically arranged spherical rollers **4** work the musculature in the areas of the neck, shoulder, back and midriff in a gentle way. Through the even up and down movements of the rollers, the muscles are massaged in a beneficial way. Switching on the heat function accelerates and reinforces the positive massage effect. The heat is emitted via the two side wings in the back section.

The unique combination of intensive vibrating massage, kneading massage and heat radiation makes the application of the roll massage cushion effective and relaxing. The individual functions can be used in any combination. Their settings can be comfortably adjusted by means of the easy-to-handle control unit **10**.

You can enjoy your massages any time you are sitting. Simply place the roll massage cushion **RBM** over a chair or armchair with a high backrest, using the practical fasteners on the back of the cushion.

The neck support cushion **1** provides extra comfort, and the control unit **10**, when not being used, can be safely stored in the control unit pocket **5**.

### 3.1 Usage

Your **MEDISANA** roll massage cushion **RBM** is a modern device for providing massage treatment.

The use of the roll massage cushion will improve your wellbeing after a stressful day. The massage effect is particularly useful for treating tense muscles and tired tissue. The unit provides pleasant relaxation for overcoming everyday stress. The massaging can improve your well-being, and is also useful in the sport and fitness areas.

### 3.2 Operation

#### Start-up

- Place the roll massage cushion on a high chair or armchair and attach it to this by means of the fasteners on the back.
- Connect the cable of the power supply unit **2** to the power supply connector **6** on the roll massage cushion.
- Insert the plug of the power supply into the power socket. Ensure that the power supply is easily accessible. Never use another mains adapter!
- Now sit on the roll massage cushion and familiarise yourself with the controls.

#### Operating the unit

The roll massage cushion **RBM** has three functions for a specific massage treatment: vibrating massage, kneading massage by means of roller technology, and heat. Each of these functions can be used individually or in combination with the others. Do not use the unit for more than 15 minutes at a time. After 15 minutes, the unit will switch off automatically. The unit is fitted with an overheat cut-off, which also switches the unit off automatically if there is any risk of it overheating. Always allow the unit to cool down fully before you use it again. Turn off the unit every time you use it and unplug the power supply unit **2**.

### Buttons on the control unit

- ON/OFF button **13** (**Power** power supply on/off)
  - Switch on : Press the ON/OFF button.  
The control LED for "Unit on" **14** will light up.
  - Switch off : Press the ON/OFF button.  
The control LED for "Unit on" **14** will go out.
  
- VIBRATING MESSAGE button **12** (**MASSAGE** in the upper thigh area on/off)
  - Switch on : Press the VIBRATING MESSAGE button.  
The control LEDs **15** and **18** will light up.  
The unit will immediately begin the vibrating massage in the upper thigh area.
  - Switch off : Press the VIBRATING MESSAGE button again.  
The control LEDs **15** and **18** will go out.
  
- VIBRATING MESSAGE mode button **9** (**MODE** Normal/Pulse)
  - Normal : Press the Mode button until the control LED for constant vibrating massage in the seat **15** (Normal) lights up.
  - Pulsierend : Press the Mode button until the control LED for constant vibrating massage in the seat **15** (Pulse) lights up.
  
- VIBRATING MESSAGE intensity button **11** (**Lo** low / **Hi** high)
  - Low : Press the Intensity button until the control LED for low massage intensity in the seat **18** (*Lo*) lights up.
  - High : Press the Intensity button until the control LED for high massage intensity in the seat **18** (*Hi*) lights up.
  
- ROLL MESSAGE button **8** (**KNEADING** kneading/shiatsu on/off)
  - Switch on : Press the ROLL MESSAGE button.  
The control LED for "Roll massage on" **17** will light up.  
The unit will immediately begin the kneading massage by means of roller technology in the neck, shoulder, back and midriff areas consecutively.
  - Switch off : Press the ROLL MESSAGE button again.  
The control LED for "Roll massage on" **17** will go out.
  
- HEAT -Taste **7** (**HEAT** heat on/off)
  - Switch on : Press the Heat button to turn on the heat.  
The heat is emitted via the two side wings of the back section.  
The control LED for "Heat on" **16** will light up.
  - Switch off : Press the Heat button again.  
The control LED for "Heat on" **16** will go out.

### 4.1 Cleaning and maintenance

- Disconnect the mains adapter from the socket before cleaning the unit.
- Only clean the roll massage cushion with a soft cloth slightly dampened with a mild detergent (e.g. soap suds). Under no circumstances should you use aggressive detergents, strong brushes, solvents or alcohol.
- When cleaning, never immerse the unit in water and make sure that no water gets into the unit.
- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

### 4.2 Disposal

Dispose of your old electrical unit in an environmentally acceptable manner when it has reached the end of its service life!

Packaging is re-usable or may be re-cycled; metal parts must be disposed of as scrap metal; plastics, electrical and electronic components must be disposed of as electro-scrap.

Consult your municipal authority or your dealer for information about disposal.

### 4.3 Technical Data

Name and model	: <b>MEDISANA</b> roll massage cushion <b>RBM</b>
Voltage supply	: 230 V~ 50 Hz
Power supply unit	: 12 V~ 2500 mA
Short operation	: 15 mins
Operating conditions	: for dry use only
Storage conditions	: in a cool, dry place
Protection class	: II 
Dimensions	: approx. 120 cm x 52 cm
Weight	: approx. 4,4 kg (without power supply)
Article No.	: 88902
EAN Code	: 40 15588 88902 8

**Right reserved to make technical and design changes during the course of continuous product improvement.**

## 5.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

### **MEDISANA AG**

Itterpark 7-9

D-40724 Hilden

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB