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## SAFETY GRABBING HANDLE WITH SUCTIONS

### ► INSTRUCTION OF REINFORCED BATH GRAB BAR WITH INDICATOR :

Your new reinforced bath grab bar with indicator offers optimum functionality and comfort. We wish you much enjoyment with it. Read these instructions carefully before assembling and using the product ! Retain the instructions for future reference ! If you pass the product on, please also pass on these product or non-observance of the information in these instructions !

### ► INTENDED USE :

Reinforced bath grab bar with indicator can be attached quickly and just as easily removed ! The suction pads are specifically designed to adhere to any smooth, pore-tight surface without drilling! Has the indicator for safety warning. Push down on the vacuum toggle lever and the handles will remain securely in place.

**The reinforced bath grab bar with indicator is only used as a balance assist only.**

**Not for body wight leverage.**

**ALWAYS MAKE SURE THE HANDLE IS SECURELY ATTACHED (THE INDICATOR HAS COME OUT AND SHOWS GREEN COLOR) BEFORE EACH USE AND ALWAYS REATTACH IF LOOSE.**

**ALWAYS TEST THE HOLDING CUP ON EACH SURFACE, EACH TIME YOU APPLY THE SUCTION PAD TO PREVENT ACCIDENTS.**

### ► INSTALLATION INSTRUCTION :

1. Check the vacuum suction discs and the surface it is to be attached to.
2. Clean the surface thoroughly.
3. Press the rubber discs firmly to the surface. The tilting vacuum levers must be loose at this point and the whole of each rubber disc must lie on the surface.
4. Ensure that the rubber discs are completely on the tile area or other surfaces so that the discs do not cover any grouting lines or other joints. The discs should be positioned at least 1/2" away from all joints.
5. First press down one tilting lever, The resistance of the negative pressure generated must be clearly felt. Make sure the green part of the indicator has appeared. The greater the resistance felt on the vacuum lever, the better the suction power of the holder have.
6. Before each use, the user or a competent assistant should check vacuum seal to ensure device is safely attached. If loose, remove entire

device and reattach per the owner's manual.

### ► REMOVAL INSTRUCTION :

1. Release both latches.
2. Slide fingertip under rubber seal at each end to release suction.

### ► CLEANING / CAUTION :

Do not use any caustic or abrasive cleaning agents or sponges. These can damage the surface. Wipe off the holder with a damp cloth and a small amount of mild detergent if necessary.

### ► MAINTENANCE :

1. Always keep the rubber discs clear of oil and dust.
2. Clean the rubber discs with standard household cleaning agents.
3. Never expose the rubber discs to sunlight for long periods otherwise they will quickly lose their elasticity.
4. Check the rubber discs for possible deterioration. If the suction cups show damage, replace immediately.

### ► DISPOSAL :

Dispose of the packing in an environmentally friendly manner. Dispose of it where valuable waste is collected. Dispose of the product through an approved disposal company or the municipal disposal facilities. Observe the current regulations. If in doubt, consult your local authority about an environmentally friendly disposal.

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