

## USAGE INSTRUCTIONS



### **Normal Curettage**

1. Use otoscope to determine cerumen location.
2. Choose the appropriate curette for the type of patient and cerumen.
3. Place curette into ear canal and remove cerumen.
4. Repeat otoscope exam and determine location of any remaining cerumen.



### **Otoscope Curettage**

1. Use otoscope to determine cerumen location.
2. Select one of the Youth Safe Ear Cures™.
3. Slide scope magnification to one side.
4. Place curette through otoscope and remove cerumen.

BIONIX, CeraSpoon, FlexLoop, InfantScoop, MicroLoop, AngleLoop, VersaLoop and ControlLoop are registered trademarks. Safe Ear Cures is a trademark of BIONIX. Patented - The ControlLoop®, CeraSpoon®, VersaLoop®, FlexLoop®, InfantScoop® and AngleLoop®.