Otoscope  
Otoscope  
Otoscopio  
Otoscopio  
Otoskop  
Zakotoscoop  
منظار أذن

ECLIPSE®

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We would like to thank you for purchasing an **Eclipse®** fibre optic otoscope. This medical device has been manufactured in accordance with the provisions of Directive 93/42 EEC applying to medical devices and is subject to frequent quality controls.

## INDICATIONS

This medical device is exclusively reserved for medical diagnostic purposes: examination of the auditory meatus and eardrum, or surface illumination during non-invasive examinations.

### WARNING

This device must be used as indicated.

## DESCRIPTION AND TECHNOLOGY

The **Eclipse®** pocket fibre optic otoscope makes precise examinations possible thanks to the quality of its illumination system and of its handling.

The optical fibres optimally focus and transmit the light beam, providing direct, homogeneous and reflection-free illumination of the auditory meatus and eardrum, even in difficult to access zones.

The outer part of the screw-on otoscope head is made from polypropylene, while the inside is made from ABS. It is fitted with a wear-resistant chromium-plated brass mount onto which the ear speculum can be easily and quickly fitted and removed.

The acrylic observation magnifier, with a magnification factor of 2.5, can swivel to the side to allow the insertion of instruments.

To perform pneumatic otoscopy, connect the blower to the air inlet port on the side of the otoscope's head.

The battery-containing handle, equipped with a metal clip, is made from high-quality, tough and impact-resistant aluminium.

## INSTRUCTIONS FOR USE

### Precautions for use

The otoscope head must be fitted with a speculum before insertion into the auditory meatus.

### Fitting batteries

Unscrew the bottom of the otoscope handle and insert two LR6 AA alkaline batteries into the battery compartment, matching the (+) and (-) polarity markings.

**Fitting speculums**

To perform an ear examination, the otoscope must be fitted with a disposable or reusable speculum.

Position the speculum on the otoscope head. Align the inner pin with the bayonet slot. Rotate the speculum clockwise fully. To remove the speculum, rotate it counter-clockwise.

**Start-up**

To turn the otoscope on or off, push the black switch above the handle clip forwards or backwards.

**Magnifier rotation**

To use instruments, swivel the magnifier to the left or right.

**MAINTENANCE****Replacing the bulb**

Unscrew the otoscope head from the handle. Next, remove the bulb from the bottom of the head. Insert the new bulb by simply pushing it fully into its housing.

**Replacing the batteries**

Unscrew the bottom of the otoscope handle. Remove the two spent batteries and insert two new LR6 AA 1.5V alkaline batteries, matching the (+) and (-) polarity markings.

**UPKEEP AND CLEANING RECOMMENDATIONS**

Regularly cleaning your otoscope will ensure optimum optical performance.

- Do not immerse the otoscope in liquid, do not sterilise.
- Clean the outside of the otoscope head with a soft cloth, or with a cloth soaked in a 70% isopropyl alcohol solution or mildly soapy water. To clean the inside of the head, use a dry or alcohol-moistened cotton bud.
- Disposable speculums should never be reused. Cleaning them could damage their end piece, leading to a risk of patient injury.
- Reusable speculums can be cleaned with a disinfectant solution.
- Keep the otoscope away from excessive heat or cold, oils and solvents.
- Avoid exposing the otoscope to direct sunlight.

**GUARANTEE**

The Eclipse® otoscope is guaranteed against manufacturing defects (parts and labour) under normal conditions of use, from the date of purchase (invoice date).

- Head and handle guaranteed for **1 year**,
- Speculums and bulb not guaranteed.

The guarantee does not apply in the event of handling non-compliant with instructions, improper use, modifications made by yourself or by an unauthorised third party, accidental damage and normal wear.

During the guarantee period, any otoscope parts that present manufacturing defects shall be replaced and/or repaired free of charge, after returning the device to Spengler's after-sales service.

In the event of a guarantee claim or of repairs, please return the complete otoscope in its original packaging, accompanied by the dated invoice.

**Contact: authorised Spengler distributor.**

## TECHNICAL DATA SHEET

<b>Head</b>	
<b>Lighting</b>	optical fibres
<b>Bulb</b>	vacuum packed 2.5 V (0.5A – screw: M 3.5 x 0.35 mm)
<b>Materials</b>	outside: polypropylene, inside: ABS speculum holder: chromium-plated brass bulb holder: chromium-plated brass
<b>Blower connector</b>	for pneumatic otoscopy
<b>Swivelling magnifier</b>	
<b>Magnification</b>	factor 2.5
<b>Material</b>	acrylic
<b>Handle with clip</b>	
<b>Material</b>	aluminium
<b>Power supply</b>	2 x LR06 AA 1.5 V alkaline batteries
<b>Accessories provided</b>	
	<ul style="list-style-type: none"> <li>• disposable speculums (pack of 10): Ø 4 mm and Ø 2.5 mm</li> <li>• 2 x LR06 AA 1.5 V batteries</li> <li>• soft storage pouch</li> </ul>
<b>Overall length (otoscope)</b>	
	16.5 cm
<b>Total weight (otoscope)</b>	
	137 g (with batteries) 90 g (without batteries)
<b>Dimensions (box)</b>	
	250x100x38 mm (Lxwxh)
<b>Total weight (boxed)</b>	
	300 g
<b>Guarantee</b>	
(normal conditions of use)	<b>1 year</b> (head and handle) Consumables (speculums and bulb) not included

Compliant with European directive 93/42 EEC applying to medical devices

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